HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49 (0) 81 52 / 38 - 0
Fax +49 (0) 81 52 / 38 - 2 02
E-Mail: info@heine.com · www.heine.com
med 3106 2014-05-26

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HEINE i-View®

General Conditions of Warranty

Instead of the statutory warranty time period of 2 years, HEINE will grant a guarantee of 5 years from the date of the consignment of the goods ex works, concerning its equipment (excluding disposables, e.g. bulbs, single-use articles, and rechargeable batteries). Date of consignment means that HEINE hands the goods over to the transport carrier, freight forwarder or any other person designated by the Customer for the transport of the goods without loading the collecting vehicle.

The guarantee covers irreproachable workmanship, on condition of the proper use of the equipment and the observation of the operating instructions. During the warranty- and guarantee time period, errors and deficiencies arising on the equipment will be rectified free of charge, in so far as such are evidenced by defective materials, processing and/or constructional errors. Should buyer complain of a material deficiency during the warranty time period, then the onus of proof is always to be on the orderer, that the product was defective already upon receipt of the goods. The statutory warranty and the guarantee do not apply to loss or damage caused by wear and tear, negligent use, the non-employment of original HEINE components and/or spares (in particular bulbs, as these have been especially developed for HEINE instruments in accordance with the following criteria: colour temperature, useful service life, safety, optical quality and performance). The statutory warranty and the guarantee do not apply to interventions by persons not authorised by HEINE or when the operating instructions are not observed by the customer. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for replacement of loss or damage, which are experienced otherwise than directly on the HEINE product itself, are hereby excluded.

Bought from

date

Instructions for use

ENGLISH

HEINE® i-View

Before using the HEINE® i-View, read these instructions carefully and keep them in a safe place for future reference.

Safety information

Range of applications: The HEINE® i-View is designed for use exclusively with HEINE instruments and light sources.

General Safety: Use only HEINE original replacement parts and accessories.

Safety in use: To avoid possible damage to the eyes, avoid looking directly at the sun or a bright light! The adaptors in illustrations 1+2 must be assembled as shown. HEINE declines responsibility for any assembly defects caused by the optician.

Do not unscrew the set screws. They are factory-set for the appropriate working distance.

Description: The HEINE® i-View and the HEINE S-Guard® are already assembled by HEINE completely. Attach the HEINE® i-View to the headlight using the adaptors which are already assembled by HEINE, too. The ease of adjustment of the joints shown in illustrations 1+2 can be adjusted with the screwdriver supplied.

Adjustment and Operation

Adjusting the nose bridge (A): This can be bent into position to suit the anatomy of your nose. By bending the bridge supports you can vary the distance between the lenses and the eyes. This avoids contact between your eyelashes or cheekbones and the frame or corrective lenses.

Changing the nose pad: The nose pad is attached to the S-Guard by a screw. To change the nose pad, unscrew the screw and remove the old nose pad.

Fitting optional corrective lenses: Please consult an optician before having corrective lenses fitted. Please show the optician the attached "Instructions for specifying corrective lenses". To fit the corrective lenses (B), insert one side of the snap-in frame (2) into the groove in the S-Frame. Next, insert the other side of the snap-in frame in the second groove.

To remove the snap-in frame with corrective lenses, reverse the above procedure, avoiding damage to the lenses.

Changing protective lenses: Hold the S-Frame as in (D) and bend the lens in the direction shown and pull the lens with an audible click out of the frame. To replace, follow the procedure shown in (E) and (F) until the lens clicks into place.

If the standard protective lenses are too close to your cheekbones, you can either fit the small protective lenses or adjust the nose bridge (A).

Swivelling the loupe: The HEINE® i-View can be swivelled upwards with one hand. Use the sterilizable swivel levers to do this (see accessories). This movement makes it possible to observe the illuminated field under investigation directly without touching the setup of the headlight.

Instructions for specifying corrective lenses:

When specifying and fitting corrective lenses, be sure to take into account the following:

- The magnifying optics should be attached to the frame to suit your usual working posture.
- Prismatic errors should be corrected.
- The corrective lenses are mounted at an angle to the optical axis.

Check the correction by means of an eye test with the magnifying optics and the corrective lenses fitted.

The total spherical correction is usually max. +/- 5 dpt. Ask your optician or ophthalmologist!

The corrective lenses should be placed as closely as possible to the eyepieces of the magnifying optics.

Cleaning and Maintenance

General: The plastic protective lenses of the S-Guard splash guard have an anti-reflex coating. Clean them carefully to avoid mechanical or chemical damage.

Manual Cleaning: Clean the S-Guard and the i-View optics mount with the enclosed HEINE microfibre cloth. We recommend the HEINE cleaning spray C-000.32.542 as a cleaning agent.

Warning: Immersion disinfection, machine cleaning/disinfection, ultrasound cleaning and sterilisation are not permitted.

Specification

Protective lenses: Plastic, anti-reflex coated
i-View and S-Guard: Nickel-free

Fig. 1

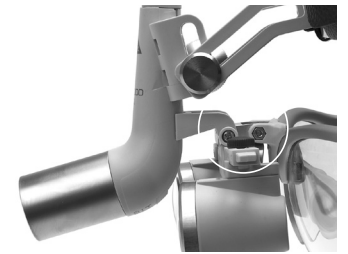
Fig. 2